



Cleaning and Handling Instructions

Dignitana understands that each hospital has its own cleaning procedures to follow for equipment, however, it is not possible for Dignitana to perform evaluations for each cleaning product that potentially can be used. Dignitana therefore recommends hospitals follow the cleaning recommendations outlined below. If other cleaning products or procedures are used Dignitana cannot guarantee the quality of the product.

DigniCap Silicone Cap

There are two recommended methods to clean the silicone cap:

1. Germicidal disposable wipes (non-bleach) wipe for 2 minutes
2. 70% alcohol or alcohol wipes

DigniTherm Click Cap

There are two recommended methods to clean the neoprene cap:

1. Germicidal wipes (non-bleach) wipe for 2 minutes
2. 70% alcohol or alcohol wipes

The cleaning recommendations for The DigniCap® Scalp Cooling System were reviewed and cleared by the FDA. Upon review it was found that the risk with multiple uses after cleaning is acceptable and is comparable to other similar nonsterile medical devices with similar cleaning instructions or mechanical cleaning (e.g. reusable blood pressure cuffs).

Handling

The DigniCap should be handled carefully and stored individually in a dry and well-ventilated place. Be sure not to store the DigniCap in a box with other items that could damage the cap sensors or a location where the cap sensors could be damaged by misuse.